

Sage Untreated Suction Toothbrush with Sage Oral Solution

REF 6977-BP



Packaging information

Contents:

- (1) Untreated Suction Toothbrush, 7 mL Sage Oral Solution, (1) Applicator Swab.

Use:

Suction Toothbrush: Aids in the removal of dental plaque, debris and oral secretions.

Instructions:

Suction Toothbrush: Instructions: Burst solution packet with thumbs. Peel package open. Attach Suction Toothbrush to suction line. Brush teeth and oral cavity for 1 minute. To suction, place thumb over port. Rinse to clear tubing after use. Discard according to hospital protocol and local regulations. Use Applicator Swab if additional cleansing is needed.

Sage Oral Solution Burst Packet: Do not rinse with water or other mouthwashes, brush teeth, or eat immediately after using Sage Oral Solution. Do not swallow. To minimize discoloration of teeth, brush and floss daily. STORE ABOVE FREEZING (0°C or 32°F).

Caution:

Do not allow patient to bite down on the oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children or unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking/aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

Not made with natural rubber latex.

Specifications

Product name	Sage Untreated Suction Toothbrush with Sage Oral Solution
Product code	6977-BP
Description	Single-use suction system for patients at risk for aspiration pneumonia
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Length of Suction Toothbrush	18.4 cm
Length of Applicator Swab	15.9 cm
Length of each bristle	8 mm
Amount of solution in each burst packet	7 mL
Case quantity	100

Features and benefits

Suction Toothbrush

Ultra-soft nylon bristles	• Gently removes dental plaque, debris, and oral secretions known to harbor respiratory pathogens
Over 3.100 bristles or "contact points"	• Cleans and removes plaque
Three suction ports	• Avoids clogging and keeps open path for debris
Available with suction handle	• Provides variable suction control and allows quick tool changes
Connects directly to standard suction lines	• Helps provide maximum suction strength
Angled handle	• Provides cleansing of the back of the oral cavity

Sage Untreated Suction Toothbrush with Sage Oral Solution

Sage Oral Solution ingredients

Water (aqua)
Alcohol
Glycerin
PEG-40 sorbitan diisostearate
Flavor (aroma)
Chlorhexidine digluconate
Sodium saccharin
Blue 1 (CI 42090)



Sage Products LLC
3909 Three Oaks Road
Cary, IL 60013 USA
800 323 2220 ; +1 815 455 4700

stryker.com

SMACC # 28858
23191C-UK
Copyright © 2021 Stryker
02/2021

This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Q-Care. All other trademarks are trademarks of their respective owners or holders.

The products depicted are CE marked in accordance with applicable EU Regulations and Directives.

Solutions including Alcohol-free Mouthwash, Comfort Rinse-free Shampoo Cap, Corinz Oral Rinse, Essential Bath, Incontinence Clean-Up Cloths, Impreva Bath, Mouth Moisturiser, and Sage Oral Solution comply with EC Cosmetic Regulation 1223/2009.

Medical devices including Oral Swabs, Sage Prevalon Heel Protector, and Sage Turn and Position (TAP) System are Class I medical devices (CE).

Sage suction medical devices including Suction Toothbrushes, Suction Swabs, and Yankauer are class IIA medical devices (CE 2797).

This document is not intended for distribution outside EU and EFTA.