

Pocket

The POCKET line presents a single use container for both sharps and cutting waste disposal. The Pocket container is equipped with a patented double opening system and can be used by just one hand. The double opening includes one side with needles and insulin pen removal device, and the other side with a large opening for entire syringe or other sharps item disposal.

The Pocket range offers attractive styling, innovative and secure usability for a personal sharps disposal unit, and is a flat design which allows easy transportability and fitment in bags or pouches amongst a user's medication (eg insulin pen, syringes etc) in comparison to the more utilitarian designs currently available.

Available Sizes: 0.4L

1. Lid Opening




Press the tongues on both sides of the container

2. Lid Closure



Temporary closure: Press down on the tongues on both sides of the container

3. Permanent Lid Closure

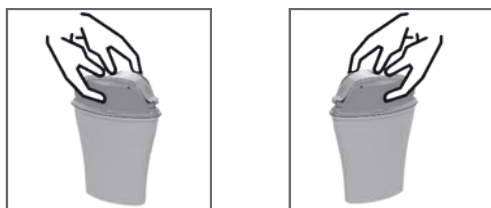

Permanent closure:

- Press firmly on one of the 'PUSH' buttons
- Press firmly on the other 'PUSH' button with 2 fingers

4. Needle Removal



Respect the maximum filling level indicated on the container

Disposal Method

Dispose in accordance with facility or local protocols

Caution:

- Do not place hands in bin
- Keep container upright
- Do not empty or re-use



Technical Specifications

Disposable and perforation resistant single use containers. Double opening system: needles and insulin pen removal / entire syringe disposal. The lid has a temporary and a permanent closure.

Capacity: 0.4L

Material: Polypropylene

No emission of any harmful gas during the incineration process

	POCKET04L
Capacity (L): Nominal Volume	0.4
Capacity (L): Real Volume	0.4
Capacity (L): Available Volume	0.35
Max load cap. (kg)	0.2
Weight (g)	105
Base Dimensions (cm)	8.6 x 3.9
Top Dimensions (cm)	12.4 x 6
Height (cm)	16.5
# Units/Carton	70



Biological hazard

Complies with ISO 23907-1, which provides guidance on the selection and recommendations for the use of this type of container. Waste must be disposed of in accordance with current laws and regulations.